<i>Elektrická odsávačka materského mléka</i> • NÁVOD K OBSLUZE	<b>CZ</b>	3-11
<i>Elektrická odsávačka materského mlieka</i> • NÁVOD NA OBSLUHU	<b>SK</b>	12-20
<i>Electric breast pump</i> • USER MANUAL	<b>EN</b>	21-29
<i>Elektromos mellszívó</i> • HASZNÁLATI ÚTMUTATÓ	<b>HU</b>	30-38
<i>Laktator elektryczny</i> • INSTRUKCJA OBSŁUGI	<b>PL</b>	39-47
<i>Elektrische Milchpumpe</i> • BEDIENUNGSANLEITUNG	<b>DE</b>	48-57

mimi

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# eta 1306

## INSTRUCTIONS FOR USE

Dear customer, thank you for purchasing our product. Keep these instructions including the receipt and, if possible, the box with the internal packing.

### I. SAFETY WARNING

- Before the first use, read the instructions for use carefully, look at the picture and store the instructions for use for the future. Consider the instructions for use as a part of the appliance and pass them on to any other user of the appliance.
- Check whether the data on the type label correspond with the voltage in your socket. The power cord plug has to be connected to the properly connected and grounded socket according to the national standard.
- This product can be used by children at the age of 8 years and older if they are supervised or if they were instructed regarding use of the appliance in a safe way and if they understand possible risks. User cleaning and maintenance must not be performed by children if they are younger than 8 year and unsupervised. Children younger than 8 years must be kept out of reach of the appliance and its power cord.
- The device may be used by persons with reduced physical or mental abilities or lack of experience and knowledge only if they are under supervision or they have been instructed about using the appliance safely and understand the potential dangers.
- Children must not play with the appliance.
- Never use the appliance if its power cord or plug is damaged, if it does not work properly, if it fell down and was damaged or if it fell to water. In this case take the appliance to a special service to check its safety and proper function.
- **CAUTION:** Do not use the appliance with a program, time switch or any other part that turns the appliance on automatically as there is a risk of fire if the appliance is not covered or placed properly.
- When the appliance is in operation, prevent pets, plants or insects to get in contact with it.
- **Do not plug the fork of the supply cord into the power outlet and do not unplug the cord with wet hands or by pulling by the supply cord!**
- The appliance is intended for home use and similar (in shops, offices and similar workplaces, guests in hotels, motels and other residential environments, in facilities providing accommodation with breakfast, in agriculture). It is not intended for commercial use!
- Do not use the appliance in an environment saturated with explosive or flammable steams.
- Thanks to double layer technology, the most of the outer surface of the electric kettle is insulated against heat. Therefore, you can touch it with your hand, even after boiling. Nevertheless, avoid contact with the outlet of the kettle during and after boiling and watch for the steam escaping from the kettle!

- Never put the power cord on hot surfaces or do not let it hang over the edge of a table or a work top. Plugging or pulling the power cord e.g. by children can result in tilting over or drawing the appliance down and serious injury!
- The power cord must not be damaged with sharp or hot objects, open fire, it must not be sunk into water or bent over sharp edges.
- Use this appliance only for the purpose for which it was designed as specified in this user's manual. Never use the appliance for any other purpose.
- The producer is not responsible for damage caused by improper use of the appliance (e.g. **burning, scalding, fire**) and does not apply in the case of non-compliance with the safety instructions above.
- The appliance has no charging source of its own, it is only designed for charging via USB (an adapter with USB output with output voltage of 5.0—12.0 V). Before making any connection, please pay attention to instructions of the manufacturer of the respective power source.
- If the appliance is used in a bathroom, it must be disconnected from the mains after use by unplugging the power cord from the wall socket, because the appliance poses a risk if near to water, even when switched off.
- Do not use USB ports on other devices (e.g. computers, laptops, tablets, mobile phones, etc.) to power the appliance. Such devices may not be dimensioned for the current consumption of the appliance and it could damage them.
- Prevent hanging the power adapter cord loosely over desktop edge where children may reach it.

## Important safety instructions

### Warnings

- Do not use in the shower.
- Please do not twist or knot the connection pipe.
- If the main machine falls into water, do not touch it, but unplug it immediately.
- The parts which are in direct contact with breast milk are personal items, do not share.
- Please keep the USB cable far away from hot surfaces.
- If the product, adaptor or USB Cable is damaged, it must be replaced by a professional recommended by the manufacturer, to avoid danger.

### Danger

- Do not put the main machine close to a fire to heat it up.
- Do not put the main machine or USB cable into any liquid.
- Don't charge or discharge near a fire or in an environment with temperatures over 45 °C.

## Precautions

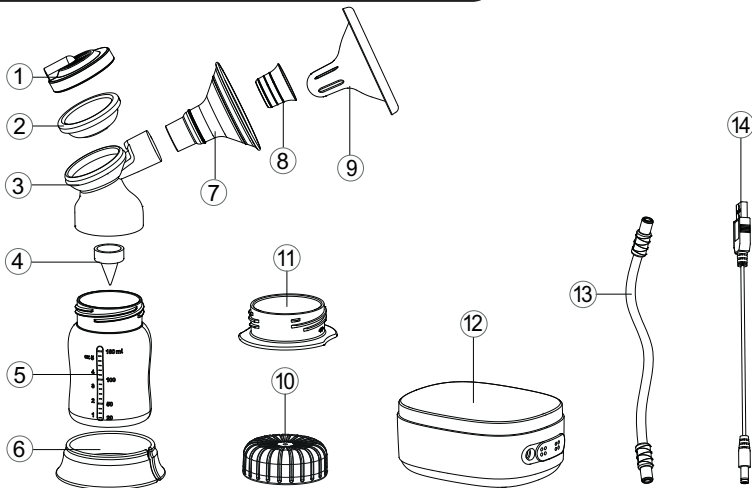
- Please charge and start before use.
- Please clean and sterilise components before use. Wash immediately after use to keep all components dry. Don't clean or sterilise the main machine or the USB cable with water.
- Keep the product out of reach of infants.
- Do not tilt or overfill the bottle when in use (do not exceed the maximum scale).
- Please turn off the power when not in use.
- When disassembling or assembling, make sure to assemble each part as described. When disassembling and cleaning, be careful not to lose some of the small parts, as it won't function properly.
- Adjust the suction level to your personal and comfortable level, start and finish pumping at this level.
- Before sterilising, please disassemble parts of the product; After sterilising, please cool the components for about 20 minutes before it is assembled for use. Otherwise, the product may be permanently damaged.
- Do not over-pump as it can cause an injury to your breast.
- In case of any pain caused by the use of this product, please seek medical help immediately.
- Please do not use this machine to start production of milk. The breast pump is a tool to help postpartum mothers ease the pain of breasts swelling and prolong the time of breastfeeding. Some new mothers, suffers from blocked mammary glands (partial reason is congenital papilla invaginated), or insufficient milk production, if this is the case, make sure to get a professional breast exam to verify that the mammary gland isn't blocked and that the breast milk is normal, only then purchase a breast pump and use it.
- This product is only used for the purpose described in this User Guide.

## Lithium battery use precautions

- The accumulator must not be removed when the appliance is normally used.
- Dismantling, impacting, extruding, or throwing into fire is prohibited.
- Do not expose to high temperatures.
- Please do not use when it is bulging.
- Do not disassemble batteries. Some components of lithium batteries are flammable and harmful. They can cause damage, fire, rupture or explode.

- Do not disassemble, open, chop or pierce the battery or battery pack with anything metallic.
- Do not over charge batteries.
- Before discarding appliances, remove the batteries.
- Lithium batteries can only be replaced by authorised service providers. Replacing the wrong type of batteries can cause explosions. When the batteries are removed, the appliances must be switched off.
- Only remove the accumulator if the appliance is to be discarded (see article VIII. ECOLOGY).
- Let the accumulator cool down prior to charging.
- During charging or regular operation, the appliance heats up, which is a normal phenomenon, and not a reason for a complaint.
- To extend the lifespan of the accumulator, we recommend discharging the accumulator completely 2x a year by normal use and then charging it fully again.
- The appliance should be kept fully charged even when not in use. It helps to extend the life of the battery.

## II. DESCRIPTION OF THE APPLIANCE

1. Upper cap
2. Silicone diaphragm
3. Pump body
4. Valve
5. Bottle
6. Bottle base
7. Breast shield

8. Silicone Insert
9. Pump cover
10. Bottle cap
11. Pump body cap
12. Pump motor
13. Silicone tube
14. USB cable




- ⏻ **Power** button: Power on/off button (\* Please charge before the first use. Press and hold for 2 sec. to starting up. It will be turn off automatically 30 mins after last operation by the default setting.
- ⏸ **Pause** button: The motor pause by clicking the pause button, click again, the motor continues to work.
- Ⓢ **Bionic** button: Click the bionic button to start the bionic mode, the icon **S** is on, 5 levels adjustable.
- ⬇ **Level up** button: Press once, and suction level increase by one.
- ⬆ **Level down** button: Press once, and suction level decrease by one.
- ⊞ **Mode switch** button: Click it to switch the stimulation and expression mode. By default the breast pump will enter into expression mode

after 2 mins of stimulation mode. When choose stimulation mode, the icon ♥ is on, 1-5 levels adjustable. Choose the expression mode, the icon ♀ is on, 1-9 levels adjustable.




Long press mode switch button to start up the **memory function**, the **M** icon light on, it will record the level of stimulation and expression mode in use. The settings will be restored at the next startup.

### III. PRODUCT ASSEMBLING

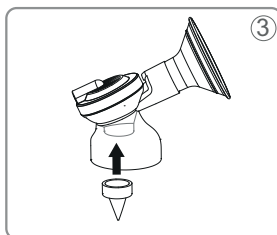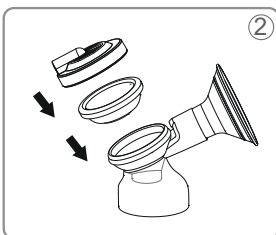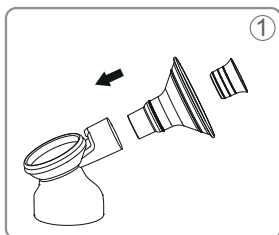1. Insert the breast shield **7** into the pump body **3**, put the silicone insert **8** into the breast shield and press down to ensure a perfect fit.
2. Insert the diaphragm **2** into the pump body **3** and make sure it is seamlessly connected to pump body. Then press the pump body upper cap onto the diaphragm as shown.
3. Insert the valve **4** into the bottom end of pump body and press to make it tightly joint.

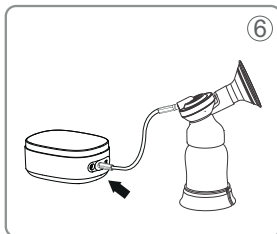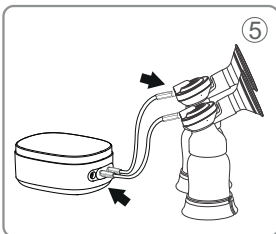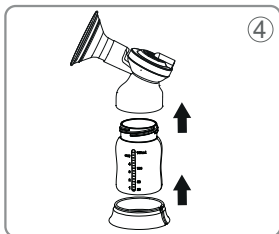
4. Screw the pump body onto the milk bottle and fix it on the bottle base as shown.
5. For double pumping, insert two silicone tubes into the two assembled sets and at the same time to both inputs on the motor unit.
6. For single pumping, connect the set with a silicone tube to one of the inlets of the motor unit. Leave the unused inlet capped with the half of cap.

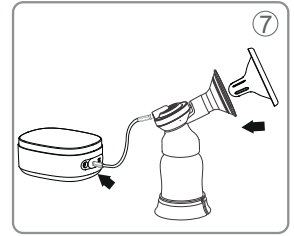7. If you do not want to use the assembled set immediately, put on the pump cover. Take it off before use and put it back on after suctioning.

**Note:** Check all the components are correctly assembled to ensure a perfect seal before each use.

#### Power adapter (not included)

- Input: 100 - 240 V~50/60 Hz, Output: DC5 V/2 A).
- Before the first use or if the appliance was not used for a long time, charge it completely.
- Connect the power adapter to the pump motor and insert the plug into an electrical outlet
- The device can only be charged when the device is switched off and connected to the adapter.
- Charging takes approximately 3 hours, and during charging the indicator light flashes , which also shows the charging status.

## IV. INSTRUCTIONS FOR USE

- Find a comfortable position and place your feet on a small stool while expressing breast milk, elevating your feet help maintain a correct sitting position and avoids putting pressure on your waist.
- Put the assembled breast shield in the middle of your nipple, ensuring there's a perfect seal.
- Press the power button  and holding for 2 seconds, it will be starting with level 1 (before activating the memory function, it will start with level 1). Press  and  to adjust the proper level during using.
- Press the button  to switch to the expression mode when the breast milk comes. (the breast pump will enter into the expression mode after 2 mins of stimulation mode automatically, and it will turn off after running for 30 minutes.)
- In use, long press , the  icon is on, the memory function start up (It will load the last levels of record automatically the next time it is used).
- How to find out the most comfortable level: please start with the low level, and increase gradually until you feel a mild discomfort (not pain), then slightly reduce the level of expression or stimulation.
- After use, please click the power button , then the indicators is off and the appliance is power off. Please unplug when not in use.
- After using, please disassembly and clean all components. (Except for motor, USB cable and silicone tube which can not be washed, these three parts only can be wiped by a clean cloth).

## V. MILK STORAGE

Storage location	Temperature	Shelf life
Room temperature	19°C - 26°C	4 hours - 6 hours
Refrigerator	4°C	72 hours - 8 days
Freezer	-15°C	2 weeks
Independent freezer of a double door refrigerator	-18°C	3-6 months
Special freezer	-20°C	6-12 months

The table is applicable to a mature healthy infant and for reference only. Please contact hospital or other professional for more info..

## Tips

- Breast milk will expand after thawing, don't fill the feeding bottle or milk bag with milk completely, and please reserve 1/4 space for expanding.
- Please mark the milk pumping time on milk bottle or milk bag.
- To save the milk's nutrition, please thaw the frozen milk in refrigerator or put the milk container under running warm water which temperature has to be lower than 37 °C. After thawing, please reheat the bottle /storage bag with milk in hot water at 50 °C (Users can use a bottle warmer to thaw frozen milk as well).
- Thawing breast milk in a microwave oven or with high temperature heater is not recommended.
- Please shake milk bottle/bag to re-mix the separated milk before feeding.

## VI. MAINTENANCE

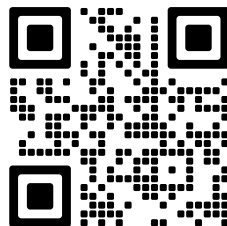
**Before any maintenance, disconnect the appliance from the electricity supply network by pulling out the plug of the power supply from the el. plugs! Do not use harsh and aggressive cleaning agents (e.g. sharp objects, scrapers, thinners or other solvents)!**

- Please wash and sterilise the parts that are in directly contact with breast milk, before and after use.
- Do not soak the main machine and USB cable in water when cleaning it, wipe it with a clean cloth.
- Please do not clean and sterilise the connector pipe. This part is a ventilation pipe, it won't get into contact with milk. Only clean its surface. (water in the pipe will cause to permanent damage of the unit)
- Diaphragm and valve are made of a silicon material, which is safe to use with food. Do not sterilise for a long time to avoid aging or deformation.
- When assembling, be sure to assemble each part in the right order. You can sterilise milk bottle, breast cushion, pump body, diaphragm, cup, valve, and pump cover. Except for the USB cable and connect pipe.
- Take care when cleaning the valve and the silicone diaphragm. If it gets damaged your breast pump can't be used.
- The pump body cannot be heated or sterilised in a microwave oven.

**More extensive maintenance or maintenance requiring intervention in the inner parts of the appliance must be carried out by professional service!**

**Failure to comply with the instructions of the manufacturer will lead to expiration of the right to repair!**

The information regarding warranty and product repairs you can find on the website [www.eta.cz/servis-eu](http://www.eta.cz/servis-eu).

## VII. TROUBLESHOOTING

Trouble	Troubleshooting method
Feel painful during use	<ul style="list-style-type: none"> <li>• Heat pack on the breast for 5 minutes before use.</li> <li>• Start from 1 level and increase the level step by step, find a appropriate level for you.</li> <li>• Please stop pumping and consult with a professional if the above methods cannot solve the problem.</li> </ul>
Not working	If the breast pump still doesn't work even if it is connected with the power adapter, please contact the distributor or designated maintenance shop.
Low suction or no suction	<ul style="list-style-type: none"> <li>• Please recheck if all components installed correctly, make sure the breast shield, Anti-flow silicone diaphragm, valve and tube etc. are connected firmly.</li> <li>• Make sure both ends of the tube is inserted into the motor unit and upper cap well.</li> <li>• Check if the valve damaged.</li> </ul>
No milk out when suction up	Be careful and ensure the papilla is aligned to the breast shield with no gap in between, pumping start from the low level repeatedly.

## VIII. ENVIRONMENT

Printed signs of materials used for manufacture, packing, components and accessories, as well as their recycling, are on all the pieces if their dimensions enable it. The symbols on the product or in the accompanying documentation mean that the used electric or electronic products must not be disposed together with municipal wastes. Please, hand them over to the collection points designed for their correct disposal, where they will be accepted free of charge. By correct disposal of this product you can help to preserve valuable natural resources and help to prevent possible negative impacts to the environment and to human health, which could be the results of incorrect waste disposal. For further details ask the municipal authorities or the nearest collecting point. Penalties can be charged for incorrect disposal of such wastes, according to national regulations. If the appliance is to be put out of operation definitely, we recommend cutting off the feeder cable, to disable its use.

### Taking out the battery

Only take out the battery from the appliance after it is fully discharged (i.e. let the appliance in operation until its motor stops), and after the end of the appliance's service life! The battery should not be removed and is not accessible during normal operation! With use of a suitable tool demount the appliance body and replace the accumulator. However pay attention to prevent any cut injury.

## IX. TECHNICAL DATA

Voltage	5 V = 2A
Battery voltage (V)	3,7 V =
Battery Voltage	2500mAh, 9.25Wh
Stimulation Frequency	40 - 210 cycles/min
Stimulation Power	15 - 165 mmHg
Expression Frequency	30 - 110 cycles/min
Expression Power	75 - 300 mmHg
Dimensions of the motor unit (mm)	132 × 103 × 59
Power supply	Input:100-240V~50/60Hz, Output: DC5V/2A

The manufacturer reserves the right to change the technical specifications and accessories for the respective models.

The capacity of the product battery decreases over time. Decreasing of battery capacity occurs during normal use of the appliance and is considered usual wear and tear of the appliance. Decreasing battery capacity is not a defect of the appliance.

**WARNINGS AND SYMBOLS USED ON THE APPLIANCE, PACKAGING OR IN THE INSTRUCTIONS MANUAL:**

HOUSEHOLD USE ONLY. DO NOT IMMERSE IN WATER OR OTHER LIQUIDS.

TO AVOID DANGER OF SUFFOCATION, KEEP THIS PLASTIC BAG AWAY FROM BABIES AND CHILDREN. DO NOT USE THIS BAG IN CRIBS, BEDS, ARRANGES OR PLAYPENS. THIS BAG IS NOT A TOY.

It is not allowed to dispose it together with communal waste

Symbol indicates a WARNING

Please read the instructions manual

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© DATE 18/04/2024
Service: Oúmega electric GmbH, Servicezentrale, Industriering 2, D- 04626 Schmölln.