DEUTSCH

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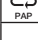
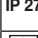
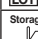
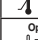
ENGLISH

Signs and symbols

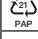
The following symbols are used in these instructions for use, on the packaging and on the type plate for the device and accessories:

	Caution
	Follow instructions for use
	Type BF applied part
	Disposal in accordance with EC Directive WEEE (Waste Electrical and Electronic Equipment)
	Manufacturer
	Dispose of packaging in an environmentally friendly manner
IP 27	Protected against foreign objects equal to or greater than 12.5 mm and against temporary immersion in water.
LOTI	Batch designation
	Permissible storage and transport temperature and humidity
	Permissible operating temperature and humidity
	The CE labelling certifies that the product complies with the essential requirements of Directive 93/42/EEC on medical devices.

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– part 3: Performance of compact electrical thermometers (non-predictive and predictive) with maximum device and the European Standard EN60601-1-2 (in accordance with CISPR 11, IEC 61000-4-2, IEC 61000-4-3 and IEC 61000-4-8), and is subject to particular precautions with regard to electromagnetic compatibility. Please note that portable and mobile HF communication systems may interfere with this unit.

Important safety information – retain for later use.
In order to make optimum use of all the merits of the clinical thermometer, you should carefully read the operating instructions before using it, retain the instructions for later use and also keep them accessible for other users.

- The thermometer is intended to measure the temperature of the human body.
- The thermometer is only designed for the measuring area on the human body stated in the instructions for use.
- Children should not use the thermometer without supervision.
- Before each use, check the thermometer for signs of damage or wear. Damaged or worn thermometers should not be used.
- Always wait for the acoustic signal which indicates the minimum time required for measurement. Discuss the temperature with your family physician.
- The thermometer contains sensitive electronic parts. Therefore, protect it from blows, bending, high temperature or direct exposure to sunlight.
- Using the thermometer in strong electromagnetic fields such as next to a mobile phone, can cause a malfunction.
- When the thermometer is switched on, it carries out a self-test. Therefore the accuracy of measurement need not be tested at regular intervals.
- If the temperature is very abnormal, please contact your family doctor promptly
- This device meets the requirements of the EU Directive 93/42/EEC concerning medical devices, as well as those of the Medizinproduktegesetz (German Medical Devices Act), the European Standard EN 12470-3: Clinical thermometers

• The accuracy of measurement is most reliable and most accurate. Guide the tip of the thermometer carefully 2 to 3 cm into the anus.

Measuring methods:
Temperature measurement in the anus (rectal)
This method of measurement is most reliable and most accurate. Guide the tip of the thermometer carefully 2 to 3 cm into the anus.

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Memory
The last measured value will be stored automatically. Press the ON/OFF button to display this value. The thermometer first carries out a self test. Most the last stored measured value is displayed for approx. 2 sec and then the reference value of 37 °C and "Lo" is shown on the display. The thermometer is ready to take a measurement and the previously stored measurement value is automatically replaced by the new measurement value.

Changing the battery
The clinical thermometer contains a long-lasting alkali-manganese battery (type LR 41, 1.5 V) for approximately 2 years. The battery needs to be replaced when all digital reading appear flashing on the display screen. Release the battery compartment cover by gently pressing on the catch. Remove the old battery and replace it with a battery of the same type. The + sign on the battery must point upwards. Then close the battery compartment, making sure that the catch clicks into place.

The empty, completely flat batteries must be disposed of through specially designated collection boxes, recycling points or electronics retailers. You are legally required to dispose of the batteries.
The codes below are printed on batteries containing harmful substances: Pb – Battery contains lead, Cd – Battery contains cadmium, Hg – Battery contains mercury. These batteries contain lead (Pb) and mercury (Hg).

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time of measurement, in spite of the acoustic signal. If temperatures over 37.8 °C are measured, a fever alarm sounds (short beeping tones, 10 seconds long). If temperatures higher than 42.9 °C are measured, the symbol for "H °C" will appear. To prolong the life of the battery, please switch off the thermometer by briefly pressing the on/off key after the measurement. Otherwise the thermometer will automatically switch off after 10 minutes.

Notes on handling batteries

- If your skin or eyes come into contact with battery fluid, flush out the affected areas with water and seek medical assistance.
- Choking hazard!** Small children may swallow and choke on batteries. Store the batteries out of the reach of small children.
- Observe the plus (+) and minus (-) polarity signs.
- If a battery has leaked, put on protective gloves and clean the battery compartment with a dry cloth.
- Protect the batteries from excessive heat.
- Risk of explosion!** Never throw batteries into a fire.
- Do not charge or short-circuit batteries.
- If the device is not to be used for a long period, take the batteries out of the battery compartment.
- Use identical or equivalent battery types only.
- Always replace all batteries at the same time.
- Do not use rechargeable batteries.
- Do not disassemble, split or crush the batteries.

Cleaning/Disinfection
The device must be cleaned and disinfected before and after each use. The thermometer is 100% water-proof. For the purpose of cleaning or disinfection, it can be dipped into water or disinfected without any difficulties. Under no circumstances should the thermometer be boiled! Use one of the following disinfectants in accordance with the manufacturer's instructions. Other cleaning agents or methods could impair the function of the thermometer or damage it.
Name of the agent: Isopropanol 70% Pharmacy; Gigesept FF; Lysoformin; Asseptisol.

Storage/Disposal
When you are not using the thermometer, please retain it in the original protective cover.

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